

END-TO-END SOLUTIONS for SARS-CoV-2 (COVID-19) Screening



RESZON COVID-19 RAPID ANTIBODY TEST & ANTIGEN TEST

CE IVD FOR PROFESSIONAL USE ONLY, NOT FOR HOME TESTING.



RESZON COVID-19 RAPID ANTIBODY TEST

FAST SCREENING of COVID-19 within 15-20 minutes

Test Description


This is an immunochromatographic assay designed for the qualitative detection of specific IgM and IgG antibodies against SARS-CoV-2, in human whole blood, serum or plasma. It is intended to be used by the professionals as a screening test and as an aid in the early diagnosis of SARS-CoV-2 infection.

Platform	Immunochromatographic Assay
Format	Cassette
Detection	Specific IgM and IgG antibodies against SARS-CoV-2
Specimen	Whole blood, serum or plasma
Specimen Volume	20 µl
Assay Time	15-20 minutes
Shelf Life	18 months

Procedures (Simplified Version)




Add 20 µl of whole blood, serum or plasma to the sample well. Make sure there is no air bubble.

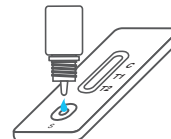

Add 1 drop of buffer to the same sample well. Read result within 15 minutes.



Result Interpretation

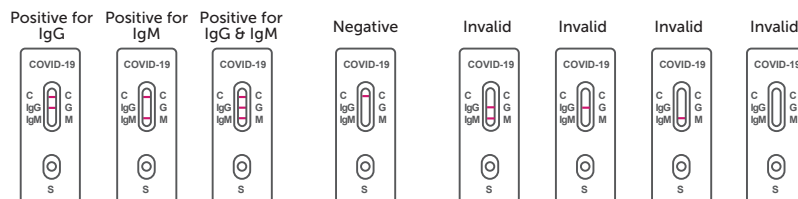

Ordering Information

Cat. No	Product Description	Test Specimen	Product Format	Intended Use	Storage Temp	Packing Size
RCV-RD0101	Reszon COVID-19 Rapid IgG/IgM Test	Whole blood, serum or plasma	Cassette	Coronavirus disease COVID-19	4-30°C	25 T/ Kit
RCV-RD0101B	Reszon COVID-19 Rapid IgG/IgM Test (with disposable dropper, alcohol swab, lancet)	Whole blood, serum or plasma	Cassette	Coronavirus disease COVID-19	4-30°C	25 T/ Kit

COVID-19



Reszon Diagnostics International Sdn. Bhd. 201001033978 (917901-W)

A member of Revongen Corporation

Headquarters

Revongen Corporation Center, Level 17, Top Glove Tower, No. 16, Persiaran Setia Dagang, Setia Alam, Seksyen U13, 40170 Shah Alam, Selangor, Malaysia

+6 03 3359 1166

+6 03 3358 0303

info@reszonics.com

www.reszonics.com

Reszon Diagnostics

■ FOR EXPORT ONLY ■

RESZON COVID-19 RAPID ANTIGEN TEST


QUICK & EARLY detection of COVID-19 infection

Test Description


This is an immunochromatographic assay designed for the qualitative detection of specific SARS-CoV-2 antigen in human nasopharyngeal swab specimen. It is intended to be used by the professionals as a screening test and as an aid in the early diagnosis of SARS-CoV-2 infection.

Platform	Immunochromatographic Assay
Format	Cassette
Detection	Specific SAR-CoV-2 antigen
Specimen	Nasopharyngeal swabs
Assay Time	15-20 minutes
Shelf Life	18 months

Procedures (Simplified Version)

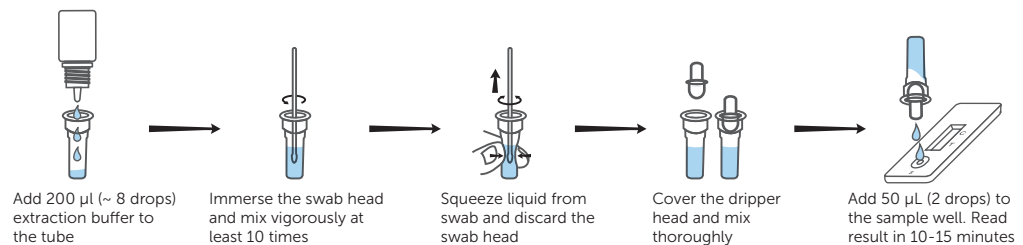

Result Interpretation

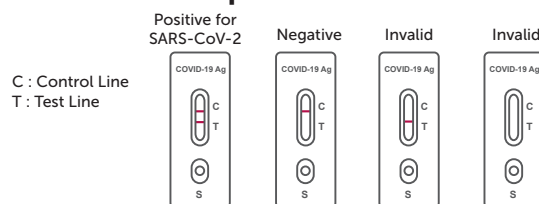

Ordering Information

Cat. No	Product Description	Test Specimen	Product Format	Intended Use	Storage Temp	Packing Size
RCV-RD0201	Reszon COVID-19 Rapid Antigen Test	Nasopharyngeal swab sample	Cassette	Coronavirus disease COVID-19	4-30°C	25 T/ Kit
RCV-RD0201B	Reszon COVID-19 Rapid Antigen Test (with Nasopharyngeal Swab)	Nasopharyngeal swab sample	Cassette	Coronavirus disease COVID-19	4-30°C	25 T/ Kit



Reszon Diagnostics International Sdn. Bhd. 201001033978 (917901-W)

A member of Revongen Corporation

Headquarters

Revongen Corporation Center, Level 17, Top Glove Tower, No. 16, Persiaran Setia Dagang, Setia Alam, Seksyen U13, 40170 Shah Alam, Selangor, Malaysia

+6 03 3359 1166

+6 03 3358 0303

info@reszonics.com

www.reszonics.com

Reszon Diagnostics

■ FOR EXPORT ONLY ■

COVID-19