

# SURE CHECK® HIV Self-Test



# SURE CHECK® HIV Self-Test



## SURE CHECK® HIV Self-Test

One Drop of Blood is All You Need

A single-use in-vitro diagnostic self-test that detects antibodies to HIV-1 and HIV-2 in fingerstick whole blood. The test is intended for use by untrained lay users in a private setting as a self-test to aid in the diagnosis of HIV infection.



### EASY TO USE

- Quick blood collection; needs only 2.5 µl of blood
- Unique Barrel Design: Allows direct transfer of blood from finger to test device



### CONVENIENT

- Low hands on time, no bottles, no mixing, no pouring
- Innovative test design prevents accidental spilling



### TRUSTED & PROVEN

- Over 13 million tests used professionally
- High Accuracy: Sensitivity: 97%; Specificity: 100%\*

\*Refer to product insert for more details

# MOST COMPLETE KIT COMPONENTS COMPARED TO LEADING BRANDS

## CONTENTS IN THE BOX



## CONTENTS IN THE POUCH



## SUITABLE FOR

- Individuals who believe they may have had exposure to the HIV virus or a risk event within the last 3 months period of time who are ages 14 and older.

## RISK EVENTS

- Individuals who have sex with someone who is HIV positive or whose status you do not know
- Sexually active individuals, regardless of their sexual orientation or gender
- Individuals who use shared needles or syringes
- Individuals who perceive themselves as at risk and have never undergone HIV screening or been diagnosed with HIV

# TEST FOR HIV IN 3 EASY STEPS WITH SURE CHECK® HIV Self-Test

**1 PREPARE**

FILL TIP OF TESTING DEVICE WITH BLOOD

**2 RUN**

POSITION TEST DEVICE VERTICALLY ABOVE TEST STAND

1<sup>st</sup> SNAP  
2<sup>nd</sup> SNAP  
3<sup>rd</sup> SNAP

PUSH HARD THROUGH THE FOIL YOU WILL FEEL 3 SNAPS

**3**

CHECK FOR PINK/PURPLE FLOW IN 1 MINUTE

PINK/PURPLE FLOW ✓  
NO PINK/PURPLE FLOW ✗

IF NO PINK/PURPLE FLOW, PUSH TEST DEVICE FURTHER DOWN

**4 READ RESULTS**

WAIT 15 MINUTES

DO NOT READ THE RESULTS AFTER 20 MINUTES

SCAN ME!  
VIDEO

**NEGATIVE RESULTS**

← CONTROL  
← TEST

C T

If test shows **ONLY** the Control Line, your test result is **NEGATIVE**.

Test again in 3 months.

**POSITIVE RESULTS**

← FAINT LINE CONTROL → ← DARK LINE TEST →

C T

If test shows **BOTH** the Control Line and Test Line, your test result is **POSITIVE**.

Visit your nearest HIV Testing Center or Health Care Professional.

**INVALID RESULTS**

← CONTROL → ← TEST →

C T

If your results do not match negative or positive, the result is **INVALID**. It is necessary to run another test.

Visit your nearest HIV Testing Center or Health Care Professional.

## Ordering Information

Cat. No.	Product Description	Test Specimen*	Product Format	Intended Use	Intended Use	Storage Temp.	Packing Size
60-9508-M	SURE CHECK® HIV Self-Test	WB	Sampler with buffer vial	Detection of antibodies to HIV Type 1 and 2	For Self-Testing	8-30°C	1 T/ Kit

Remarks: \*WB – Whole Blood

Distributed by



Manufactured by



Reszon Diagnostics International Sdn. Bhd. 201001033978 (917901-W)

info@reszonics.com www.reszonics.com

Doc No.: MM-02-03-08-EN  
Rev. No.: 01  
Effective Date: 2024.03.14

testnow

SCAN ME!  
REPORT RESULT



Reszon Diagnostics